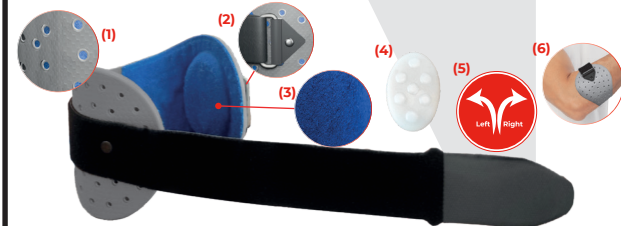
- 1 The main body of the product is aluminum. The aluminum body is coated with a special material from the thermoplastic elastomer (TPE) class that is flexible and soft enough to suit the anatomy. Thanks to the ergonomic shape and design, it fits well on the area where it is applied.
- 2 The product is easily applied by passing the strap through the buckle and pulling it to this belt, the required pressure can be adjusted.
- 3 The terry cloth on the inner surface has a sweat absorbing structure.
- 4 Silicone pad which has optimum hardness provides comfortable support.
- 5 Can be applied to the right or left side.
- 6 Can be used for treatment of lateral epicondylitis (tennis elbow) and medial epicondylitis (golfer's elbow). Whichever of these both problem the product will be used, it is applied to the area where the silicone pad puts pressure on that area.

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EN • In cases of lateral epicondylitis, the epicondylitis bandage is applied such a way that the silicone pad on the inside of the bandage comes to the lateral side.

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EN • Extend your arm in a straight position. Apply the epicondylitis bandage from the lateral line of the elbow towards the wrist to the underside of the elbow, where your doctor indicates. If the bandage is applied in the right place, you will feel the pressure of the bandage during dorsiflexion movement. Do not apply more pressure than necessary.

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
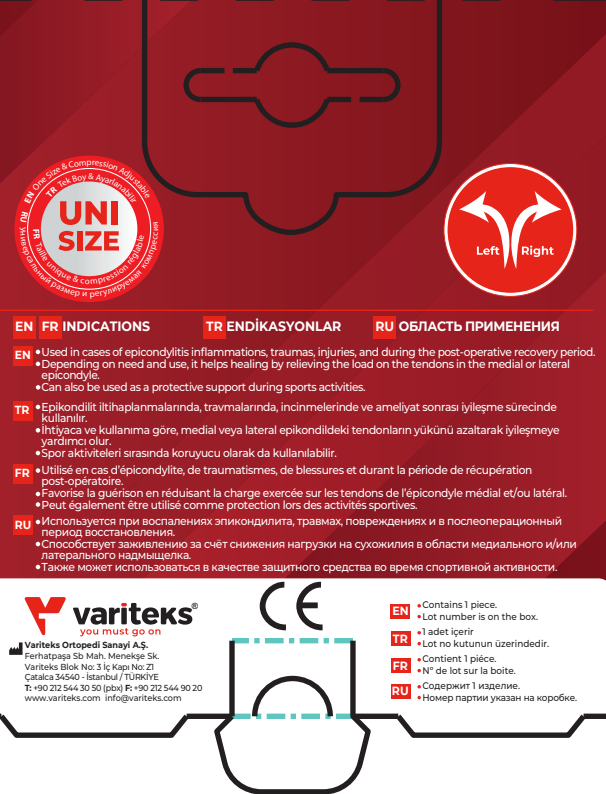
EN • Contains 1 piece.
Lot number is on the box.

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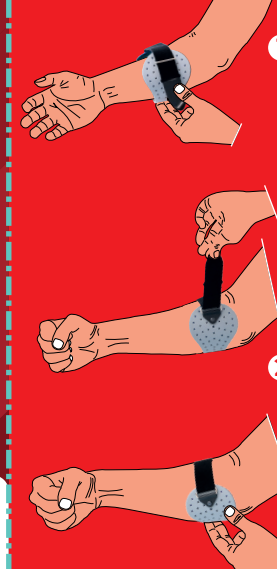- **EN** In cases of medial epicondylitis, the epicondylitis bandage is applied such a way that the silicone pad on the inside of the bandage comes to the medial side.
- **TR** Uygulama Şekli
- **FR** COMMENT APPLIQUER
- **RU** КАК ПРИМЕНЯТЬ

EN In cases of medial epicondylitis, the epicondylitis bandage is applied such a way that the silicone pad on the inside of the bandage comes to the medial side.

• **EN** Extend your arm in a straight position. Apply the epicondylitis bandage from the medial line of the elbow towards the wrist, placing just below the elbow as instructed by your doctor. If the bandage is applied correctly, you will feel the pressure it creates during flexion movement. Do not apply more pressure than necessary.

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